ventura top

Putty

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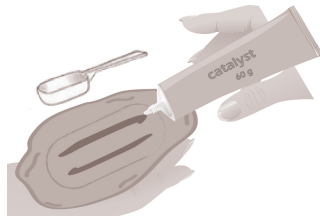
INSTRUCTIONS FOR USE

DOSAGE

Use the measuring spoon to dose the required quantity of putty and spread it out over your hand (note: for the correct dosage the putty should be level with the top of the measuring scoop). Press the rim of the measuring scoop onto the putty once for each measure dosed. For each measure of material, apply two strips of Catalyst Gel as long the measuring scoop (4 cm).

MIXING

Fold and knead the putty energetically with your fingertips for about 30 seconds, until the colour is even, without any stripes. Spread out the putty over the impression tray. The working time including mixing time is about 1 minute and 15 seconds. Put into the mouth and wait until it is set, which will take about 3 minutes and 15 seconds. The total setting time is 4 minutes and 30 seconds.

WARNING

In sensitive subjects, polysiloxanes may cause irritation or other allergic reactions.

CLEANING AND DISINFECTING THE IMPRESSION

After rinsing well under running water, the impression can be immediately disinfected. When using disinfectants, always follow the manufacturer's instructions.

CASTING MODELS

The impression must be cast in between 30 minutes to 72 hours after hardening. The material is compatible with all quality commercially available chalks. We recommend the following gypsum: Type IV Ventura Superdie Rock and Type III Ventura Stone.

WARNING / CAUTION

- Before making the corrective impression, remove all remaining traces of oral fluids by rinsing thoroughly and drying.
- The times given are for mix starting at 23°C. High temperatures or overuse of catalysts will speed up hardening, whereas low temperatures or insufficient use of catalysts will slow it down.
- Close all hardening substances properly after use.
- Prevent catalyst from coming into contact with skin and eyes; wash well with soap and water in the event of accidental skin contact. In the event of eye contact, rinse thoroughly with running water and seek medical attention. If substance is ingested, seek medical attention immediately.
- Do not stain garments.

GALVANISATION

The impressions may be galvanised with copper or silver.

CONSERVATION

Guaranteed for 36 months at 5° to 27°C (41° – 80°F).

FOR DENTAL USE ONLY

Instructions for use issued in April 2016 (2016-04).

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