ventura top lab 85

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EN HIGH PRECISION CONDENSATION SILICONE (POLYSILOXANES) FOR THE DENTAL LAB

INSTRUCTIONS FOR USE

VENTURA TOP LAB 85. Polysiloxane which may be mixed with gel catalyst, characterised by good final hardness (> 80 Shore A), high stability and capacity for reproduction of details and resistance to deformation.

USE

- Masks for the manufacture of temporary prosthetics; addition or replacement of elements in total or skeletally anchored prosthetics.
- Removable gingival masks
- Partial counter-moulding in the muffle for partial or total prosthetics.
- Anchorage in prosthetics repair.
- Occlusal adjustment key for positioning in the articulator.
- Duplication of total prostheses and plaster models without use of insulation.
- Bases for working with photo-polymerising compound resins.

INSTRUCTIONS FOR USE

Dosage:

Use the measuring spoon to take the required quantity of **VENTURA TOP LAB 85** and spread it on the palm of your hand (note: the measuring spoon must be level). Press the edge of the measuring spoon onto the mass once for each spoonful used.

For every spoonful used, apply two strips of **VENTURA TOP CATALYST** as long as the measuring spoon, which is 4 cm.

Mixing / use:

Fold the mass over on itself and knead it energetically with your fingertips for about 30 sec. until colour is even, without stripes. Now begin to model the mass as required. You have about 2 minutes to work with the mass, including mixing time, at room temperature if gel catalyst is used; with liquid catalyst, the time is reduced by about 30 sec. Hardening takes about 6 min.

TECHNICAL DATA

Mixing time	30"
Mixing time	2'
Setting time	6'
Shore-A hardness	80
Compressive deformation	<1%
Elastic recovery	99%
Linear dimensional variation	-0,10%

WARNINGS / PRECAUTIONS

The times given are calculated from the start of mixing at a temperature of 23°C (73°F). High temperatures or excessive use of catalyst will speed up hardening. Low temperatures or insufficient catalyst will slow it down. Close indurents properly after use. Do not stain clothing. Use of gloves is recommended. Avoid contact of catalyst with skin and eyes. In the event of accidental contact with the skin, wash well with soap and water. In the event of contact with the eyes, rinse well with water and seek medical attention. If ingested, seek medical attention immediately.

STORAGE

Guaranteed for 36 months if stored correctly at temperatures between 5° and 27°C (41°-80°F).

*The times given are calculated from the start of mixing at 23°C (73°F).

Instructions for use issued in October 2016 (2016-10).

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