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1. DESCRIPCIÓN

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EN INSTRUCCIONES FOR USE

1. DESCRIPTION

Ventura bite is a silicone elastomer of the vinyl-polysiloxane type whose reticulation takes place based on a polyaddition reaction. The reticulated product's mechanical properties and reproduction of detail are excellent. Ventura bite presents a high thixotropy and low contraction. It does not adhere to organic resins or gypsum.

2. INDICATIONS

Accurate occlusal registrations.
Key material for needle point registration.
Production of small model segments.
For applications that require hard bite registrations.

3. PREPARATION OF THE DISPENSING GUN

- Load the cartridge by sliding it vertically through the leads provided for this purpose.
- Press the lever until the plunger rests on the cartridge.

4. MIXING

Unscrew the cartridge cap by turning it 90° (1/4 of a revolution), remove it and discard it, as it cannot be reused. Extrude impression material until the same amount comes out from the cartridges bores and then mount the appropriate mixing tip, blocking it by means of a 90° turn (1/4 of a revolution). For a more precise dispensing, a bite registration oral tip may be placed on the mixing tip

5. MAKING THE IMPRESSION

Apply bite registration material according to the desired technique (occlusal or buccal). Extrude material first onto areas where teeth have been prepared keeping the end of the mixing tip immersed in the material to avoid the formation of air bubbles and ask the patient to bite. The material should be placed directly on the teeth within 30 seconds from starting the mix and kept in occlusion position for at least 45 seconds. Carefully remove the set bite registration after the recommended time. Remove all residues of the material from the mouth. Trim bite registration to ensure correct seating on model. Once you have finished the procedure, leave the mixing tip on the cartridge until the next use as it serves as a cap. Between uses, store the dispensing gun with the mixing tip facing down. Intraoral tips and mixing tips are for single use only.
For the next mix using the same cartridge, remove the used mixing tip by turning it 90° (1/4 of a revolution), insert a new mixing tip and proceed.

6. DISINFECTION

Once bite registration is removed from mouth, rinse under cold water. The impression can then be disinfected with appropriate disinfecting solutions or sprays such as 2% glutaraldehyde. Take care to follow the manufacturer's instructions for use.

7. PRECAUTIONS

- Contact with latex gloves may impair the setting reaction. Vinyl gloves are recommended.
- Do not combine with condensation curing silicones, polyether or polysulfide.
- Avoid contact of materials with eyes. In case of contact rinse eyes with plenty of water and consult a physician immediately.
- Silicone adheres to clothes from where it may be very difficult to remove, even by using chemical agents.
- Do not use after the expiry date.

8. CONTRAINDICATIONS

The substances that the product contains may cause allergic reactions to certain individuals.

9. STORAGE

In order for the product to maintain its optimum characteristics, it is advisable to store it at temperatures between 10°C and 25°C (50°F-77°F) and use it before the expiry date.

10. LOT NUMBER AND EXPIRY DATE

They are printed on each package.

11. TECHNICAL CHARACTERISTICS

Classification ISO 4823: Type 2 – Medium - bodied consistency.

Color: Blue.

Mixing ratio base : catalyst: 1 : 1.

Mixing time : Automixing.

Working time at 23 °C: 30*.

Time in oral cavity: 45**.

Setting time: 1'15***.

Linear dimensional variation at 24 hours: < 0.1 %.

Shore A Hardness at 24 hours: 93.

*These times are intended from the beginning of mixing at 23°C (73°F). Higher temperatures shorten while lower temperatures prolong these setting times.

12. FOR PROFESSIONAL USE ONLY

These materials have been developed to be used exclusively by dentists and must be handled according to the instructions for use. The recommendations included in the instructions correspond to the current state of knowledge regarding dental techniques. The manufacturer cannot be held responsible for damages caused by other uses or incorrect handling. **Caution:** Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

Instructions for use issued in June 2021 (2021-06).

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