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1. DESCRIPCIÓN

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EN INSTRUCTIONS FOR USE

1. DESCRIPTION

VENTURA IMPRESS 90 LAB is an elastic silicone (polyvinyl siloxane) for dental laboratory use. It comes in two putties with high viscosity (BASE+CATALYST), easily malleable, and which are mixed manually in a ratio of 1:1 polymerizing per addition.

Suitable for all applications (registers or molds) where it is necessary to faithfully reproduce a surface, VENTURA IMPRESS 90 LAB guarantees a high definition of details in all conditions of use. The high degree of hardness achieved by the polymerized mass, together with a high elastic recovery, make it especially suitable when greater rigidity is required and in all those cases in which it is necessary to keep the record for a long time. Surfaces obtained with VENTURA IMPRESS 90 LAB are easily scanned with the most common CAD/CAM/CIM systems.

2. APPLICATIONS

- Occlusal registration keys.
- Fronts for the realization of:
 - partial prostheses.
 - temporary crowns and bridges.
 - bars.
 - front teeth (immediate prosthesis).
 - gingival reproduction (indirect technique).
 - ceramic print.
- Control keys for metal structures.
- Control keys for diagnostic models.
- Keys for photopolymerizable resins
- Teeth Insulation when muffle.
- Support for prosthesis repair.
- Duplications in general.

3. HANDLING GUIDELINES

Dose an equal amount of BASE and CATALYST (1 : 1) preferably using the supplied measuring spoons (yellow dispenser for BASE and white dispenser for CATALYST).

Mix the two components manually, kneading them until you get a dough with a perfectly homogenic color. Next, shape the mass according to your needs, taking care not to exceed the indicated working time, which is measure at a temperature of 23 ° C. Wait at least 4' before removing the mask and proceed to further processing. Mixing ratio: 1:1
Mixing time: 30"
Working time: 1' 30"
Adjustment time: 3' 00"
Removal time: 4' 00"
Hardness (24 h): 90 Shore A
Color: White (CATALYST) Purple (BASE)

4. TIPS FOR OPTIMAL USE

For optimal and reproducible results, when using VENTURA IMPRESS 90 LAB, we recommend:

- Handle the product with disposable vinyl or nitrile gloves. If you intend to wear latex gloves, check their compatibility well before using them: contact with products containing sulfides or active nitrogen may adversely affect the setting of type A silicones (polyvinylsiloxanes with additive polymerization) such as VENTURA IMPRESS 90 LAB.
- Close the bottles tightly after use, being careful not to change the BASE/CATALYST caps and measuring cups. Cross-contamination of the two masses can trigger partial polymerization, which can damage the product to the point of making it unusable.
- Use VENTURA IMPRESS 90 LAB at room temperature as close as possible to 23 °C. Higher ambient temperatures accelerate, even significantly, working and adjustment times; on the contrary, lower temperatures slow down the mentioned times.
- Do not prolong the mixture beyond the indicated times; this can accelerate, even considerably, the working and set-up times, due to the heating that is produced by friction and contact with the hands. Do not use an amount of CATALYST (white) higher than BASE (light blue) to accelerate the setting process, as this does not affect polymerization times.
- Properly prepare the surfaces with which VENTURA IMPRESS 90 LAB will come into contact, which must be dry and free of oily residues to obtain a faithful and well-detailed reproduction.

5. ADVANTAGES

- Simple and hygienic mixing.
- Not greasy to the touch.
- Plastic consistency, high softness.
- High accuracy.
- Accurate reproduction of details.
- High elastic recovery.
- High final hardness.
- High dimensional stability prolonged over time.
- Easy cutting and milling
- Extremely low linear retraction.
- Heat resistant up to 200 °C.
- Scannable.

6. STORAGE

Store away from heat sources, at a temperature between 10 and 30 °C (50-86 °F).

7. SAFETY

When using CONPHILAB 90, we recommend: In case of contact with the eyes, rinse immediately with plenty of running water. Consult an ophthalmologist if irritation persists.

8 EXPIRY DATE

The product is guaranteed for 24 months if stored correctly. Use the product before the expiry date indicated on each pack.

Instructions for use issued in June 2022 (2022-06).

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